


## Instructions for Use

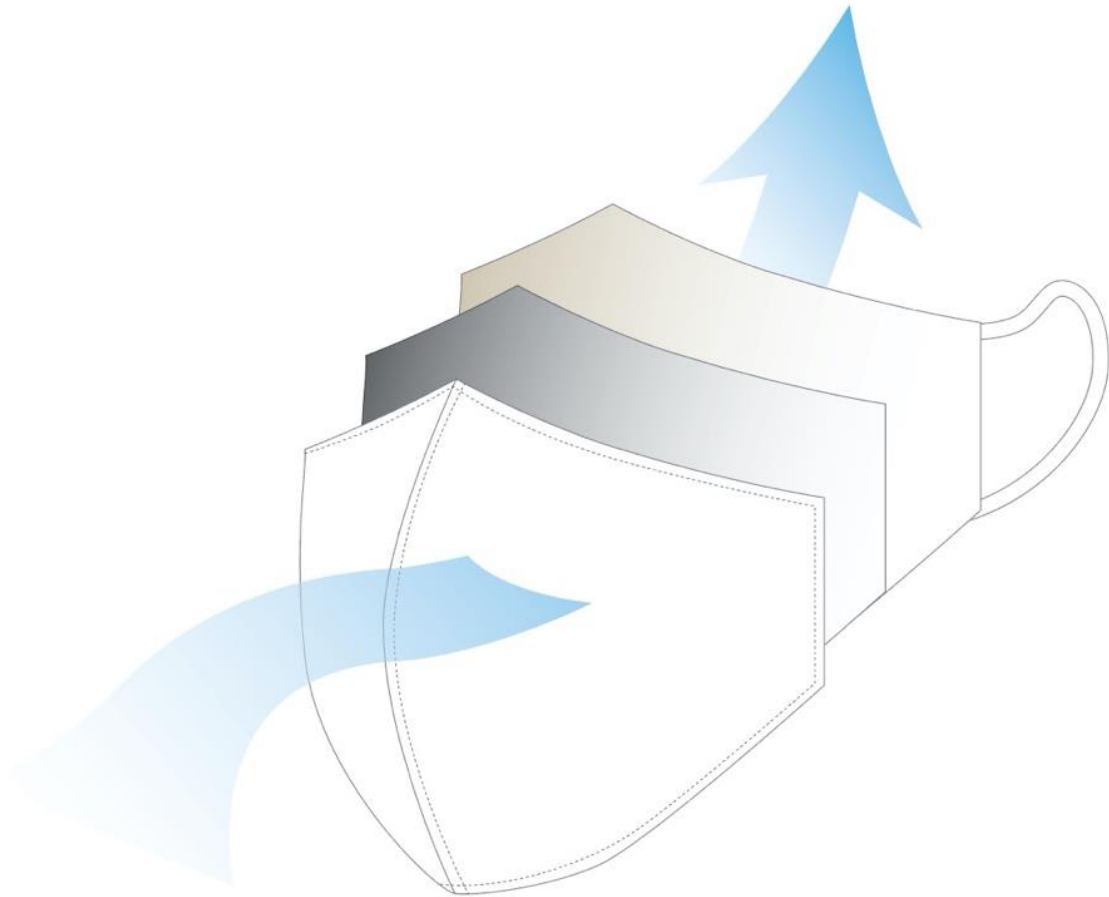

## NANO ARMOR

NANO SILVER ANTI-BACTERIA FABRIC FACE MASK

The CE mark on this product indicates that the manufacturer has declared it to be in compliance with the guidelines of the Medical Device Directive 93/42 / EEC, Medical Device Class 1 SARS-Cov-2 CORONA Maske for limited multiple use. According to KN95 (GB2626:2006)

**Purpose:**

The NANO SILVER ANTI-BACTERIA FABRIC FACE MASK is a 3-Layer Protection Mask with Nano Silver Antibacterial/Antimicrobial properties which inhibits the growth of bacteria, molds, fungi and other microbes through the release of silver ions. The mask is used to cover nose and mouth.

The mask is also available with water repellent coating, it repels up to 90% of fluids. The mask is reusable and can be machine washed up to 45 times at 40 degrees without affecting filter performance.

**Functions:**

- Antimicrobial, actively protects against viruses and bacteria
- Machine Washable up to 45 times at 40 degrees
- Economical and ecological through reusability
- Surgical mask type IIR according to DIN / EN 14683
- Medical device class I according to EWG93 / 42, MDR 98/79 EG / 2017/745, MEDDEV guidelines.

**How to use:**

Please unpack the mask from the package. Please hold the mask with both hands, each hand holding one ear loop. Open the mask with the inner side (silk) facing the nose and mouth and place the ear loops over both your ears. Check whether your nose and mouth is fully covered and adjust fitting by hand.

The CE mark on this product indicates that the manufacturer has declared it to be in compliance with the guidelines of the Medical Device Directive 93/42 / EEC, Medical Device Class 1 SARS-Cov-2 CORONA Maske for limited multiple use. According to KN95 (GB2626:2006)

### Mask Types:


Type NAWR: 3-layered with Water Repellent Coating  
Type NAWW: 3-layered without Water Repellent Coating

### Specifications:

Mask body outer layer: 100% cotton  
Mask body middle layer: 100% cotton with embedded Nano Silver silk  
Mask body Inner layer: silk

Ear loop: 95% cotton and 5% elastane, flat knitted  
Weight: 14 grams  
Reusable: washable up to 45 times manually  
(machine with net)

Available colors: White, Blue, Black, Tan

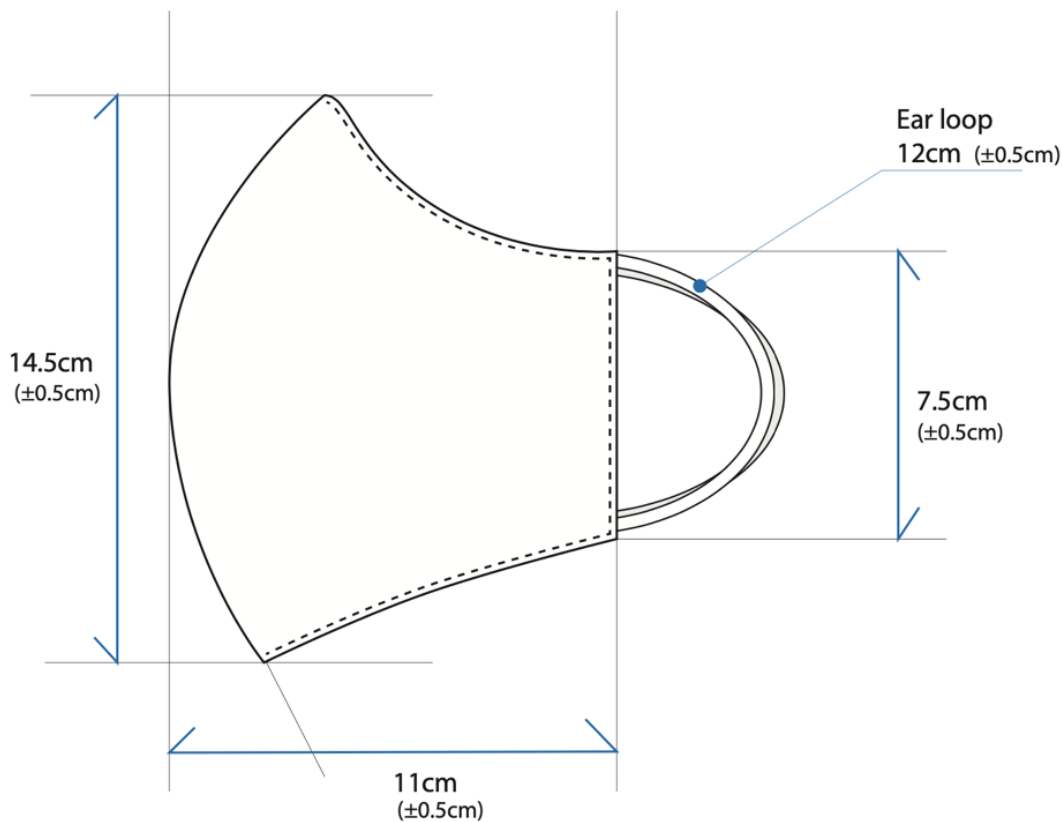

The CE mark on this product indicates that the manufacturer has declared it to be in compliance with the guidelines of the Medical Device Directive 93/42 / EEC, Medical Device Class 1 SARS-Cov-2 CORONA Maske for limited multiple use. According to KN95 (GB2626:2006)



Manufacturer:

Cosmic ME  
2-31-3 Shibashimo Kawaguchi  
Saitama 333-0848 Japan  
Tel.: + 81-48-268-9811  
[www.cme.co.jp/en/](http://www.cme.co.jp/en/)

EU Representative:

Spindler Consulting, Pizolstr. 7,  
CH-7310 Bad Ragaz / Schweiz  
[Spindler.dr@gmail.com](mailto:Spindler.dr@gmail.com)

Tel.: +49-174-6787846 / +41-762871429  
UID-Nr.:CHE-238.762.071

The CE mark on this product indicates that the manufacturer has declared it to be in compliance with the guidelines of the Medical Device Directive 93/42 / EEC, Medical Device Class 1 SARS-Cov-2 CORONA Maske for limited multiple use. According to KN95 (GB2626:2006)